# Abbott FreeGo

Bomba para Alimentación Enteral

## Manual de Uso

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# TABLE OF CONTENTS

---

INDICATIONS FOR USE.....	36
PUMP CHARACTERISTICS .....	37
WARNINGS.....	38
PRECAUTIONS.....	38
FRONT AND BACK PANELS .....	40
LCD PANEL DISPLAYS.....	41
FEEDING SETS .....	42
INSTRUCTIONS FOR USE.....	43
ALARM AND WARNING MESSAGES .....	49
PUMP DIAL AND BUTTON REFERENCE .....	53
CLEANING .....	57
PUMP ACCURACY TESTING.....	58
ACCESSORY INFORMATION .....	59
SERVICE INFORMATION.....	59
WARRANTY INFORMATION .....	59
DISPOSAL INFORMATION.....	59
SPECIFICATIONS.....	60
SYMBOLS .....	63
DISTRIBUTORS.....	64
STANDARDS.....	65
CONTACT INFORMATION.....	69

▶ **Abbott FreeGo  
Enteral Feeding Pump**

- ▶ **Operator Manual**  
For Enteral Use Only  
Not For Intravenous (IV) Use  
For Use With: S400 A58

## INDICATIONS FOR USE

---

The Abbott FreeGo enteral feeding pump is intended for hospital, acute care, long-term care, and home use. It is designed for both stationary use and ambulatory use. The pump is also designed to function safely on a commercial aeroplane.

The Abbott FreeGo pump can be used for adult and paediatric patients, provided the patients can tolerate a feeding range within the pump operational specifications.

- The flow rate range is 1 to 400 mL/hr, in 1 mL/hr increments.
- The flow rate accuracy is +/- 5%, under recommended head height (see PUMP ACCURACY TESTING).
- Pump operates against 103 kPa to 124 kPa nominal back pressure before occlusion alarm.

**If these specifications are not appropriate for a given patient, the Abbott FreeGo pump should not be used.**

# PUMP CHARACTERISTICS

---

The Abbott FreeGo enteral feeding pump is a microprocessor (computer)-controlled rotary peristaltic pump that provides for accurate, controlled enteral feeding in a safe, user-friendly system. It has a large, easy-to-read display and simple controls. The pump uses specially designed Abbott FreeGo feeding sets, each of which includes a bolus-prevention feature to provide safe, accurate control of enteral feeding.

## **The Abbott FreeGo pump offers these features:**

1. Alarm features (see ALARMS AND WARNINGS)
2. Flow-rate selection from 1 to 400 mL/hr, in 1 mL/hr increments
3. Accuracy to  $\pm 5\%$  (see PUMP ACCURACY TESTING)
4. Dose setting
5. Volume-fed accumulation
6. 24-hour battery operation at 125 mL/hr, when fully charged
7. Programmable PAUSA (HOLD) timer
8. Lock-out feature
9. System self-check
10. User-friendly controls
11. Simple setup
  - » Pump memory has automatic and indefinite retention of the following values until reprogrammed or cleared:
    - ACCUMULATED VOLUME FED/DELIVERED
    - RATE
    - DOSE

## WARNINGS

---

- **NOT FOR INTRAVENOUS (IV) USE**
- Possible explosion hazard if used in the presence of flammable anaesthetics.
- No modification of this equipment is allowed.

## PRECAUTIONS

---

- The Abbott FreeGo pump is designed to deliver liquid enteral feeding products only (standard liquid products, infant formulas, or reconstituted powder products that have been thoroughly mixed into solution).
- Exposure to magnetic fields, electrical external influences, and electro static discharges may affect the operation of any electronic medical device, including enteral feeding pumps.
- All enteral feeding pumps have the potential to bolus-feed or over-deliver, which is an important consideration in feeding volume-sensitive patients. In these patients, a volume of product no greater than four (4) times the hourly feeding rate should be in the feed container.

**NOTE:** Do not rely on the pump's alarms as a criterion for use with volume-sensitive patients.

- Confirm proper placement and function of patient's enteral feeding tube. Failure to do so may result in vomiting and/or aspiration. Verify the following before initiating a feeding:
  1. Only an Abbott FreeGo feeding set is being used.
  2. Abbott FreeGo feeding set pump insert is lightly stretched counterclockwise around rotor and lilac cassette is secured to the pump. Also ensure that the pump insert is correctly seated around the rotor.
  3. If the Abbott FreeGo feeding set is damaged in any way (e.g. broken, cut, torn, parts separated), replace the set immediately.
  4. Flow rate is set at the prescribed mL/hr.
- Abbott FreeGo feeding set and container should be replaced as needed, or at least every 24 hours, to avoid contamination problems.
- Abbott FreeGo feeding sets are for single-patient use only.


- Do not use excessive force when manually priming the Abbott FreeGo feeding set. Doing so could alter feeding accuracy.
- If the pump is dropped it should be checked by Abbott Nutrition-authorized technical personnel prior to use on a patient. Studies have shown that dropping from any angle from a height of 1 metre (3 Feet) should not damage pump operation.
- Whenever priming air from the Abbott FreeGo feeding set, make certain that the feeding set is not connected to the enteral feeding tube.
- Prior to pump startup, user should:
  1. Clean the pump (see CLEANING).
  2. Inspect the pump and AC adapter charger, and should not use the system if there is evidence of damage or tampering.
- User should verify that, during the pump startup, the pump's LCD display and audio both operate according to the Self-Check Procedure (see the first NOTE under TO START PUMP in the INSTRUCTIONS FOR USE). Do not use if the pump's operation is not exactly as described.
- Only use approved Abbott FreeGo AC Adapter Charger or 12V Automobile Charger to operate the FreeGo pump.
- There is a potential risk for entanglement with the tubing from a pump set in some patient populations such as children. These patients should be monitored during feeding as deemed necessary by their Health Care Professional.
- The FreeGo pump battery may be fully discharged when received and may require approximately 2 minutes of power prior to illumination of the AC/Mains LED indicator.


# FRONT AND BACK PANELS

---

## Front Panel

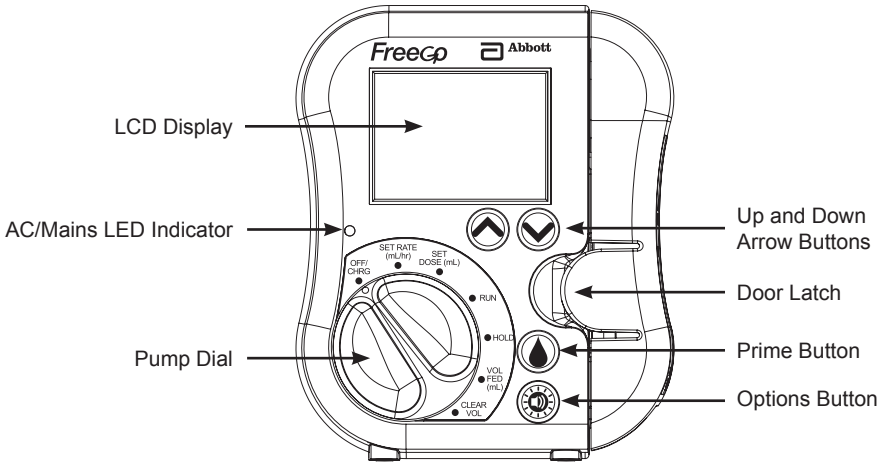
## Back Panel
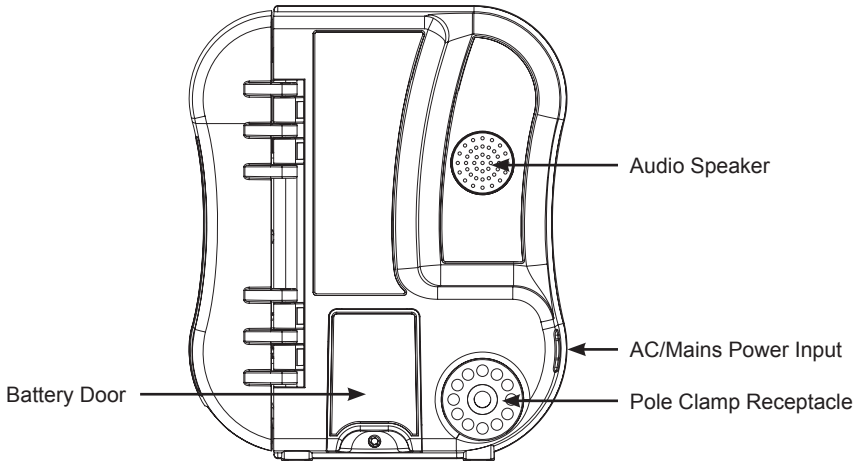
# LCD PANEL DISPLAYS

---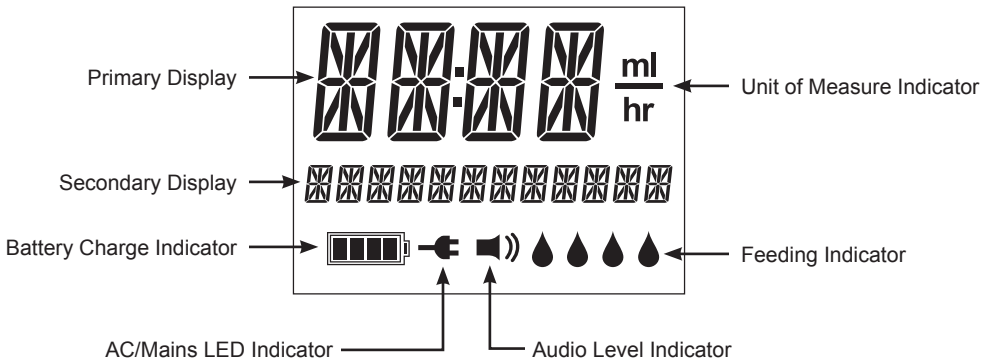

**Primary Display:** This 4-character display shows the value of the selected dial parameter. This would include the rate, the dose, or the volume fed, depending on the pump's dial position. Abbreviated alarm messages will also show here.

**Secondary Display:** This 12-character display is used to provide additional information to the user.

**Unit of Measure Indicator:** This display indicates the unit of measure (mL, hr, mL/hr) corresponding to the value displayed in the Primary Display.

**Battery Charge Indicator:** The battery charge status is indicated by battery icon located on the lower left side of the display. When operating on batteries the number of bars showing in the battery indicates the amount of battery charge remaining. When charging the battery bars scroll.


**AC/Mains Operation Indicator:** AC/Mains-powered operation is indicated by a plug icon located to the right of the battery icon, as well as an LED light above the dial.

**Audio Level Indicator:** The audio volume level setting is indicated by this icon. If the setting is at ALTO (HIGH), there will be two bars (🔊). If the setting at BAJO (LOW), there will be one bar (🔊).

**Feeding Indicator:** Four drop icons will scroll in the lower right hand corner of the display during feeding, incremental priming, or auto priming. Otherwise, the drops are not present.

# FEEDING SETS

---

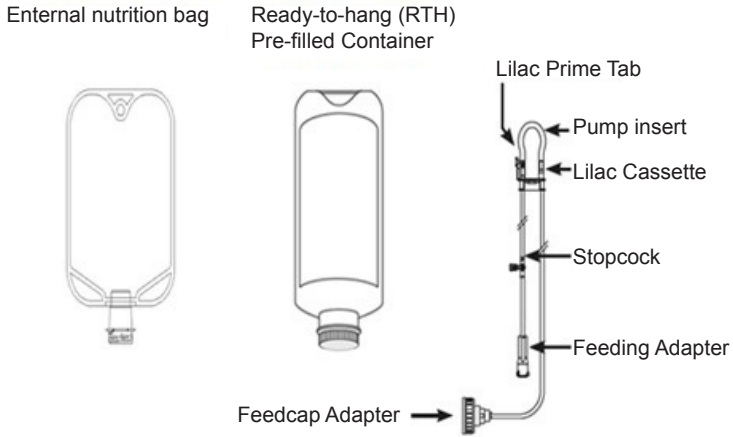**NOTE:** not shown, 220 mL RPB (reclosable plastic bottle) and Flexitainer

## ***PRECAUTIONS:***

- The Abbott FreeGo feeding set and container should be replaced as needed, or at least every 24 hours, to avoid contamination problems.
- For single-patient use only.
- Do not store for prolonged periods at extreme temperatures.
- Only Abbott FreeGo sets can be used with the Abbott FreeGo enteral feeding pump.

# INSTRUCTIONS FOR USE

---

## **TO PREPARE FEED CONTAINER:**

1. Depending on your feed container, complete ONE of the following (A, B, or C):
  - A. If feeding from a Ready-to-Hang (RTH) container, attach either the Spike or Feedcap feeding set securely onto the pre-filled enteral nutrition container.
  - B. If feeding from a RPB (reclosable plastic bottle), attach the Feedcap feeding set securely onto the pre-filled enteral nutrition container.
  - C. If feeding from a bag set, unscrew the cap on the bag set, pour in the feeding solution, and be sure to securely re-screw the cap onto the bag set to prevent accidental fluid spilling.
2. Hang the feed container (i.e. from a drip stand, from Abbott FreeGo table drip stand, or inside Abbott FreeGo backpack).

## **TO PRIME AND LOAD THE FEEDING SET:**

1. Complete ONE of the following (Option A or Option B):

### **Option A**

- » MANUALLY PRIME THE SET: Gently push the lilac prime tab on the feeding set cassette against tubing until fluid flows. Completely prime the set by allowing fluid to expel air from the tubing.
- » LOAD THE SET INTO THE PUMP: Open the door on the pump. Grasp the lilac cassette body on the feeding set and loop the pump insert around rotor stretching lightly. Gently pull down and then seat the lilac cassette into the pump. Gently press the tubing to ensure it is fully seated in slots below the cassette. Close the door firmly.

**Or...**

### **Option B**

- » LOAD THE SET INTO THE PUMP: Open the door on the pump. Grasp the lilac cassette body on the feeding set and loop the pump insert around rotor stretching lightly. Gently pull down and then seat the lilac cassette into the pump. Gently press the tubing to ensure it is fully seated in slots below the cassette. Close the door firmly.
- » AUTO PRIME THE SET: Turn pump dial to FLUJO (SET RATE) or DOSIS (SET DOSE). Press and hold the Prime Button for two (2) seconds then release. "CEBANDO" ("PRIMING") will be displayed. Auto priming will stop when the priming volume has been reached. To stop this process prior to its completion, press and release the Prime button.

- » INCREMENTALLY PRIME THE SET: There may still be a small section of air in the feeding set. Press and hold the Prime Button. Priming will begin after two (2) seconds. Release Prime Button when fluid has reached desired level.

**NOTE:** See  in PUMP DIAL AND BUTTON REFERENCE for more information on priming.

### **TO START PUMP:**

1. Turn pump dial to **FLUJO (SET RATE)**. Then select your prescribed flow rate by pressing the Up and Down Arrow Buttons while observing the entry on the display. Keeping an Arrow Button depressed will cause the scroll rate to increase.

**NOTE:** Any time the dial is turned from **OFF/CARGA (OFF/CHRG)** to any other dial position (i.e. the pump is turned on), the pump will initiate system Self-Check Procedure. During this procedure, user should **verify** that all segments on the LCD display activate and deactivate, then the pump's software version and serial number (for example, **V1:00 AF11001000**) are displayed. Simultaneously, verify that the audio beeps at high volume, then low, then high again. *If the pump has a software version later than V1:01, then the pump will also display either **NC ENABLED** or **DATA ENABLED**, at the end of the sequence described above.* Do not use the pump if the operation is not exactly as described above.

2. Turn the dial to **DOSIS (SET DOSE)** and programme your prescribed dose using the Up and Down Arrow Buttons. Keeping an Arrow Button depressed will cause the scroll rate to increase.

**NOTE:** Pump can be programmed to feed until the container is empty (**DOSIS (SET DOSE)** at **INF DOSIS** (infinite dose); see PUMP DIAL AND BUTTON REFERENCE). In this case, feeding will continue until the pump's sensors detect air in the feeding set. At that time, the pump will stop and the display will read "**NING ALIMENTO**" ("**FEED EMPTY**").

**NOTE:** Be sure that the feeding volume has been cleared from the pump's memory prior to beginning a new feeding (for instruction, see **VOL CERO (CLEAR VOL)** at step 6, below).

3. Connect the Abbott FreeGo feeding set securely to the enteral feeding tube.
4. Turn dial to **MARCHA (RUN)** to start feeding.

**NOTE:** If alarm sounds, turn pump dial to **PAUSA (HOLD)**. Correct alarm condition that has been indicated on the LCD display, then return dial to **MARCHA (RUN)** to restart feeding. For Alarm and Warning information, see ALARM AND WARNING MESSAGES.

5. To view the accumulated feeding volume delivered, turn dial to **VOL ADM (VOL FED)**.
6. To clear the accumulated feeding volume delivered, turn dial to **VOL CERO (CLEAR VOL)** (the volume will clear after a five (5) second countdown.)
7. When feeding is complete, turn the pump dial to **OFF/CARGA (OFF/CHRG)**.
8. Detach the feeding set from the enteral feeding tube.

### ***TO PAUSE FEEDING:***

In order to temporarily pause feeding, turn the dial to **PAUSA (HOLD)**. The pump's hold timer will start its countdown from 5 minutes, by default. The hold time can be manually changed in one-minute increments, from 1 to 90 minutes, by using the Up and Down Arrow Buttons. The pump's hold time resets back to this default value every time the dial is switched away from **PAUSA (HOLD)**. At the end of the countdown, an alarm will sound, but the **PUMP WILL NOT AUTOMATICALLY RESTART**. The dial must be returned to **MARCHA (RUN)** for the pump to begin feeding again. The hold timer is only meant as an audio reminder that the feeding was paused.

**NOTE:** Since feeding only occurs in **MARCHA (RUN)** mode, feeding may be temporarily paused by switching the dial from **MARCHA (RUN)** to any of the other dial settings. HOWEVER, there is the possibility of changing the feeding parameters (in **FLUJO (SET RATE)** or **DOSIS (SET DOSE)** dial mode) or clearing the accumulated delivered volume (in **VOL CERO (CLEAR VOL)** dial mode). Therefore, it is advised to only use the **PAUSA (HOLD)** function to temporarily pause a feeding.

### ***TO LOCK THE PUMP:***

In order to prevent an unintentional altering of the current numerical values in **FLUJO (SET RATE)**, **DOSIS (SET DOSE)** and **VOL ADM (VOL FED)** (via **VOL CERO (CLEAR VOL)**), the pump has a lock-out feature.

**NOTE:** The **PAUSA (HOLD)** timer can still be adjusted while the lock-out feature is activated.

**NOTE:** The pump will not prime the feeding set while the lock-out feature is activated.

1. To activate the lock-out feature, simultaneously press both the Up and Down Arrow Buttons while the dial is in either **FLUJO (SET RATE)** or **DOSIS (SET DOSE)** dial modes. The display will read "**ON BLOQUEO ACTIVADO**" ("**ON LOCK ON**").
2. To deactivate the lock-out feature, repeat the step (1) above. The display will read "**OFF BLOQUEO DESACTIVADO**" ("**OFF LOCK OFF**").

## ***USING THE DATA DOWNLOAD ACCESSORY CABLE (S412):***


**CAUTION:** The pump COMM (communication output) setting must be configured to DATA (for Data Download Accessory Cable).

**NOTE:** Pumps with software version **V1:01** are not configured for use with the Data Download Accessory Cable (S412). To check the software version of your pump, from the **OFF/CHRG** position, turn the pump dial to **SET RATE**. During the system Self-Check Procedure the pump display will show the software version. If your pump has software version **V1:01**, contact your local Abbott representative to have the pump upgraded.

To configure your pump for use with the Data Download Accessory Cable:

Press the Options Button  until COMM is displayed on the pump display. Then use the Up and Down Arrow Buttons until **DATA** is displayed.

Pump must be plugged into live AC/Mains power in order for Data Download Accessory Cable to work. Refer to the drawing below for cable and charger connections.

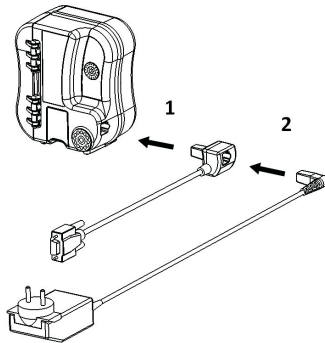To verify functionality of the Data Download Accessory Cable (S412), turn the pump dial to **RUN**. Then open the pump door, which will trigger a **DOOR OPEN** alarm. Verify the alarm is transmitted from the Data Download Accessory Cable.

### TROUBLESHOOTING:

If the **DATA** function is not working properly:

- Check to see that all ends of the accessory cable and charger are securely connected.
- Check the accessory cable for damage.
- If the cable is properly connected and not damaged, return the pump and cable for service.

## **USING THE NURSE CALL ACCESSORY CABLE (S468):**


**CAUTION:** The pump COMM (communication output) setting must be configured to **NC** (for “Nurse Call”, Nurse Call Accessory Cable).

**NOTE:** Pumps with software version **V1:01** are not configured for use with the Nurse Call Accessory Cable (S468). To check the software version of your pump, from the **OFF/CHRG** position, turn the pump dial to **SET RATE**. During the system Self-Check Procedure the pump display will show the software version. If your pump has software version **V1:01**, contact your local Abbott representative to have the pump upgraded.

To configure your pump for use with the Nurse Call Accessory Cable:

Press the Options Button  until COMM is displayed on the pump display. Then use the Up and Down Arrow Buttons until NC is displayed.

Pump must be plugged into live AC/Mains power in order for Nurse Call Accessory Cable to work. Refer to the drawing below for cable and charger connections.

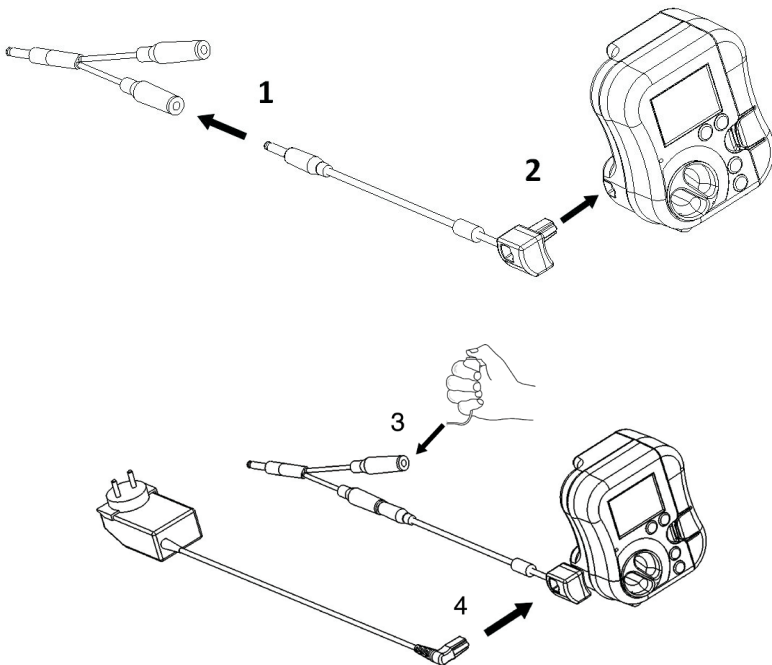**CAUTION:** Be certain to verify functionality at the nurse call station prior to connecting the feeding set to the patient.

To verify functionality of the Nurse Call Accessory Cable (S468), turn the pump dial to **RUN**. Then open the pump door, which will trigger a **DOOR OPEN** alarm. Verify the nurse station is receiving this alarm. Press the nurse call button and verify the nurse station is receiving this alarm.

**CAUTION:** The Nurse Call Accessory Cable functionality should be verified periodically.

TROUBLESHOOTING:

If the **NC** function is not working properly:

- Check to see that all ends of the accessory cable and charger are securely connected.
- Check the accessory cable for damage.
- If the cable is properly connected and not damaged, return the pump and cable for service.

# ALARM AND WARNING MESSAGES

PRIMARY SECONDARY DISPLAYS	CONDITION OR PROBLEM	EXPLANATION OR CORRECTIVE ACTION
<b>OBST</b> NO ENTRA ALIM (OCCL INTO PUMP)	Flow has stopped due to occlusion between the formula container and the pump.	<ul style="list-style-type: none"> <li>Turn dial to <b>PAUSA (HOLD)</b></li> <li><u>Remove feeding set from pump.</u> Clear obstruction; check for kinked tubing or empty container. Reinstall feeding set.</li> </ul>
<b>OBST</b> NO SALE ALIM (OCCL OUT OF PUMP)	Flow has stopped due to occlusion between the pump and the patient.	<ul style="list-style-type: none"> <li>If this alarm appeared during priming, attempt to prime the feeding set again, then connect the feeding adapter to your enteral feeding tube</li> <li>Turn dial to <b>MARCHA (RUN)</b> to begin feeding</li> </ul>
<b>ATN</b> BOMBA EN PAUSA (ATTN PUMP ON HOLD)	Hold time has elapsed.	<ul style="list-style-type: none"> <li>Turn dial to <b>MARCHA (RUN)</b> to begin feeding</li> </ul>
<b>BAT</b> CARGA BAJA (BATT LOW BATTERY)	Pump battery is nearly empty. At least 20 minutes of battery capacity is remaining from time of first alarm.	<ul style="list-style-type: none"> <li>Connect the pump to live AC/Mains power.</li> </ul>
<b>BAT</b> AGOTADA (BATT BATT EMPTY)	Pump battery is depleted. Pump has stopped feeding.	<ul style="list-style-type: none"> <li>Connect the pump to live AC/Mains power. The display will indicate "<b>CARG POR FAVOR ESPERAR</b>" ("<b>CHRG PLEASE WAIT</b>")</li> </ul>
<b>ATN</b> PUERTAABIERTA (DOOR OPEN)	The door is open while feeding or priming the feeding set.	<ul style="list-style-type: none"> <li>Turn dial to <b>PAUSA (HOLD)</b></li> <li>Close the door</li> <li>Turn dial to <b>MARCHA (RUN)</b> to begin feeding, or attempt to prime the feeding set again</li> </ul>

PRIMARY SECONDARY DISPLAYS	CONDITION OR PROBLEM	EXPLANATION OR CORRECTIVE ACTION
<b>NING</b> CARTUCHO ( <b>NO CASSETTE</b> )	Cassette is not installed or was installed incorrectly when user attempted to run pump or prime the feeding set.	<ul style="list-style-type: none"> <li>• Turn dial to <b>PAUSA (HOLD)</b></li> <li>• Insert cassette and close door</li> <li>• Turn dial to <b>MARCHA (RUN)</b> to begin feeding, or attempt to prime the feeding set again</li> </ul>
<b>ER##</b> FALLA ELECTRONICA ( <b>ERR## SYSTEM FAIL</b> )	Pump malfunction.	<ul style="list-style-type: none"> <li>• Remove pump from patient immediately and contact your local Abbott Nutrition representative</li> <li>• Turn dial to <b>OFF/CARGA (OFF/CHRG)</b></li> </ul>
— MINUTOS DE PAUSA ( <b>HOLD MINUTES</b> )	Hold time count down. The primary display shows the remaining time.	<ul style="list-style-type: none"> <li>• Pump dial is at <b>PAUSA (HOLD)</b></li> <li>• If desired, adjust the <b>PAUSA (HOLD)</b> timer to any duration (in 1-minute intervals) from 1 to 90 minutes by depressing the Up and Down Arrow Buttons</li> </ul>
<b>BLOQ</b> BOMBA BLOQUEADA ( <b>LOCK PUMP LOCKED</b> )	Lock-out feature disables <b>FLUJO (SET RATE)</b> , <b>DOSIS (SET DOSE)</b> , and <b>VOL CERO (CLEAR VOL)</b> dial settings, as well as the pump's prime features.	<ul style="list-style-type: none"> <li>• Turn dial to <b>FLUJO (SET RATE)</b> or <b>DOSIS (SET DOSE)</b></li> <li>• Depress both Up and Down Arrow Buttons simultaneously to deactivate lock-out feature</li> <li>• Repeat steps to activate</li> </ul>

PRIMARY SECONDARY DISPLAYS	CONDITION OR PROBLEM	EXPLANATION OR CORRECTIVE ACTION
<b>NING</b> ALIMENTO (FEED EMPTY)	Air is detected in the feeding set.	<p><i>If you are just beginning a feeding:</i></p> <ul style="list-style-type: none"> <li>Open the door. <u>Gently press the tubing to ensure it is fully seated in slots below the cassette.</u> Close the door firmly.</li> </ul> <p><i>Or... If there is more food to deliver:</i></p> <ul style="list-style-type: none"> <li>Turn dial to <b>PAUSA (HOLD)</b> to stop the audio alarm</li> <li>Remove feeding set adapter from your enteral feeding tube</li> <li>Refer to INSTRUCTIONS FOR USE to prime the air out of the feeding set</li> <li>Securely re-attach the feeding set to your enteral feeding tube</li> <li>Turn dial to <b>MARCHA (RUN)</b> to begin feeding</li> </ul> <p><i>Or... If you are done feeding:</i></p> <ul style="list-style-type: none"> <li>Turn dial to <b>OFF/CARGA (OFF/CHRG)</b>. This message is displayed because pump was programmed to feed until container is empty (<b>DOSIS (SET DOSE)</b> at <b>INF DOSIS</b> (infinite dose)).</li> </ul> <p>This alarm condition may be caused by electromagnetic interference (reference the Guidance and Manufacturer's Declaration - Electromagnetic immunity table in the Standards section)</p> <ul style="list-style-type: none"> <li>Move pump away from cellular phones, amateur radios, AM and FM radio broadcast, TV broadcast or other radio frequency (RF) transmitting devices.</li> </ul>

PRIMARY SECONDARY DISPLAYS	CONDITION OR PROBLEM	EXPLANATION OR CORRECTIVE ACTION
<b>FIN</b> DOSIS ADMINISTRADA (DOSE DONE)	The pump has completed delivering the programmed dose.	<p><i>If you are done feeding:</i></p> <ul style="list-style-type: none"> <li>Turn dial to <b>OFF/CARGA (OFF/CHRG)</b>, the dose has been delivered. This message is displayed because <b>VOL ADM (VOL FED)</b> has reached your <b>DOSIS (SET DOSE)</b></li> </ul> <p>Or... If there is more food to deliver:</p> <ul style="list-style-type: none"> <li>Turn dial to <b>VOL CERO (CLEAR VOL)</b> (after 5 seconds, volume will be cleared)</li> <li>Turn dial to <b>DOSIS (SET DOSE)</b>, and use Up and Down Arrow Buttons to set your remaining dose to be delivered</li> <li>Turn dial to <b>MARCHA (RUN)</b> to begin feeding</li> </ul>
<b>DIAL</b> MAL COLOCADO (DIAL NOT SET)	Pump dial is between two dial positions.  <i>For example: the dial is stuck halfway between <b>MARCHA (RUN)</b> and <b>PAUSA (HOLD)</b>.</i>	<ul style="list-style-type: none"> <li>Turn the dial to the desired dial position, directly aligning the two dots (on the dial and the dial position label)</li> <li>Continue setting up and operating pump as normal</li> </ul>
<b>CARG</b> POR FAVOR ESPERAR (CHRG PLEASE WAIT)	The battery charge is below the minimum acceptable level and the pump cannot be used until the minimum charge level is reached.	<ul style="list-style-type: none"> <li>No action can be taken.</li> <li>Pump will be available for use when the display indicates "<b>BOMB LISTA PARA USAR</b>" ("<b>PUMP READY TO USE</b>")</li> </ul>
<b>BOMB</b> LISTA PARA USAR (PUMP READY TO USE)	The battery charge has reached the minimum level and the pump can now be used.	<ul style="list-style-type: none"> <li>Turn dial to any position to clear message</li> <li>Confirm all settings are accurate</li> <li>Turn dial to <b>MARCHA (RUN)</b> to continue feeding</li> </ul>

# PUMP DIAL AND BUTTON REFERENCE

---

DIAL / BUTTON	FUNCTION
<b>OFF/CARGA (OFF/CHRG)</b>	<p><b>Primary purpose is to shut down all pump functions.</b></p> <ul style="list-style-type: none"><li>• If switched from any dial position to <b>OFF/CARGA (OFF/CHRG)</b>, the display will briefly read “<b>BOMB APAGANDO BOMBA</b>” (“<b>PUMP SHUTDOWN</b>”), and then enter OFF mode.</li><li>• If switched from <b>OFF/CARGA (OFF/CHRG)</b> to any other dial position, the pump will initiate system Self-Check Procedure. During this procedure, user should <b>verify</b> that all segments on the LCD display activate and deactivate, then the pump’s software version and serial number (for example, <b>V1:00 AF11001000</b>) are displayed. Simultaneously, verify that the audio beeps at high volume, then low, then high. <i>If the pump has a software version <u>later than V1:01</u>, then the pump will also display either <b>NC ENABLED</b> or <b>DATA ENABLED</b>, at the end of the sequence described above.</i> Do not use the pump if the operation is not exactly as described above.</li><li>• If external power supply <u>is not connected</u>, the display will be blank. If external power supply <u>is connected</u>, the display will show AC/Mains icon and battery icon.</li></ul> <p><b>NOTE:</b> If external power supply is connected, pump battery will charge in <u>any</u> dial setting.</p>
<b>FLUJO (SET RATE)</b>	<p><b>Primary purpose is to set the rate/speed (the amount of fluid volume per hour) at which feeding solution is delivered to patient.</b></p> <ul style="list-style-type: none"><li>• To program feeding rate, use Up and Down Arrow Buttons. Keeping an Arrow Button depressed will make the numerals scroll. The scrolling rate will increase (depending on length of time Arrow Button is depressed).</li><li>• Auto Prime and Incremental Prime features are functional in this dial position.</li><li>• To set the lock-out feature, simultaneously press both Up and Down Arrow Buttons. To deactivate lock-out feature, repeat procedure.</li><li>• If lock-out feature is not set, the display will read “<b>FIJE FLUJO</b>” (“<b>SET RATE</b>”), along with the current rate setting (from 1 to 400 mL/hr).</li><li>• If lock-out feature is set, the display will read “<b>BOMBA BLOQUEADA</b>” (“<b>PUMP LOCKED</b>”).</li></ul>

---

DIAL / BUTTON	FUNCTION
<b>DOSIS (SET DOSE)</b>	<p data-bbox="284 188 964 236"><b>Primary purpose is to set the dose (to limit the total amount of fluid volume) delivered during an entire feeding.</b></p> <ul data-bbox="320 264 1020 788" style="list-style-type: none"> <li data-bbox="320 264 1020 363">• To program feeding dose, use Up and Down Arrow Buttons. Keeping an Arrow Button depressed will make the numerals scroll. The scrolling rate will increase (depending on length of time Arrow Button is depressed).</li> <li data-bbox="320 384 990 483">• <i>If you wish to feed until the container is empty, set the dose to <b>INF DOSIS</b> (infinite dose). To do this, make sure the pump dial is in the <b>DOSIS (SET DOSE)</b> position then depress the Down Arrow Button until <b>INF DOSIS</b> appears on the display.</i></li> <li data-bbox="320 504 997 552">• Auto Prime and Incremental Prime features are functional in this dial position.</li> <li data-bbox="320 572 992 620">• To set the lock-out feature, simultaneously press both Up and Down Arrow Buttons. To deactivate lock-out feature, repeat procedure.</li> <li data-bbox="320 641 1005 719">• If lock-out feature is not set, the display will read “FIJE DOSIS” (“SET DOSE”), along with the current dose setting (from 1 to 9999 mL, or <b>INF</b>).</li> <li data-bbox="320 740 1005 788">• If lock-out feature is set, the display will read “BOMBA BLOQUEADA” (“PUMP LOCKED”).</li> </ul>
<b>MARCHA (RUN)</b>	<p data-bbox="284 831 818 858"><b>Primary purpose is to initiate and continue a feeding.</b></p> <ul data-bbox="320 884 992 1305" style="list-style-type: none"> <li data-bbox="320 884 833 908">• This is the <u>only</u> dial setting that operates a feeding.</li> <li data-bbox="320 928 992 1023">• As long as dial is kept in this position, feeding will continue until: <ul data-bbox="390 970 992 1023" style="list-style-type: none"> <li data-bbox="390 970 992 1023">» The volume delivered equals the programmed dose volume (when VOL ADM equals DOSIS),</li> </ul> </li> <li data-bbox="379 1043 418 1067">or...</li> <li data-bbox="390 1082 788 1106">» An alarm condition has been detected.</li> <li data-bbox="320 1118 726 1142">• Display will show the programmed rate.</li> <li data-bbox="320 1163 807 1187">• Display will scroll 4 drops in bottom-right corner.</li> <li data-bbox="320 1208 992 1305">• It is normal for rotor movement to be intermittent (e.g. starts, stops, starts, etc.) while in <b>MARCHA (RUN)</b> mode. This intermittent movement regulates the rate of flow, conserves battery charge, and does not affect the accuracy of the pump.</li> </ul>

DIAL / BUTTON	FUNCTION
<b>PAUSA (HOLD)</b>	<p><b>Primary purpose is to remind patient that the pump is temporarily paused. Pump will <u>not</u> automatically restart the feeding after the hold time has elapsed. To restart the feeding, user must turn dial back to MARCHA (RUN).</b></p> <ul style="list-style-type: none"> <li>To programme hold time, use Up and Down Arrow Buttons. Keeping an Arrow Button depressed will make the numerals scroll.</li> <li>Display will read “MINUTOS DE PAUSA” (“HOLD MINUTES”), along with the current remaining hold time (from 1 to 90 minutes). Default is 5 minutes, and the pump’s hold time resets back to this default value every time the dial is switched away from <b>PAUSA (HOLD)</b>.</li> </ul>
<b>VOL ADM (VOL FED)</b>	<p><b>Primary purpose is to display the accumulated volume of feeding solution that has been delivered.</b></p> <ul style="list-style-type: none"> <li>When this number matches the <b>DOSIS (SET DOSE)</b>, feeding will stop and an audio alarm will sound along with a visual display alarm (reading “FIN DOSIS ADMINISTRADA” (“DOSE DONE”).</li> <li>Once upper limit (9999 mL) has been reached, the display will show “- - - -” and is not accompanied by alarm. Therefore, it is critical to perform <b>VOL CERO (CLEAR VOL)</b> function (below) prior to beginning feeding.</li> <li>Display will read “VOL ADMINISTRADO” (“VOLUME FED”), along with the current accumulated fluid volume that has been delivered (from 1 to 9999 mL).</li> </ul>
<b>VOL CERO (CLEAR VOL)</b>	<p><b>Primary purpose is to allow the user to clear the accumulated volume of feeding solution that has been delivered and been stored in the pump’s memory (shown at dial position VOL ADM (VOL FED)).</b></p> <ul style="list-style-type: none"> <li>If lock-out feature is not set, the display will read “BORR EN <u>n</u> S” (“CLEAR IN <u>n</u> SEC”), where <u>n</u> represents the number of seconds before the accumulated volume delivered is set back to 0 mL. This is a countdown from 5 seconds, and each second is accompanied by a beep. When the volume has been cleared, a beep will sound for 2 seconds.</li> <li>If you do not wish to clear the accumulated volume delivered, turn the dial away from <b>VOL CERO (CLEAR VOL)</b> before the 5 seconds has elapsed.</li> <li>If lock-out feature is set, the display will read “BOMBA BLOQUEADA” (“PUMP LOCKED”).</li> </ul>

The Up and Down Arrow Buttons are used for increasing or decreasing values, respectively. They also serve to toggle between different settings (see  below).

The Prime Button is used to remove air from the feeding set. This button is functional only in the **FLUJO (SET RATE)** and **DOSIS (SET DOSE)** dial positions. While priming, be absolutely certain that the feeding set is not connected to your enteral feeding tube. See INSTRUCTIONS FOR USE.

The pump will begin auto priming if this button is depressed when the pump's sensors detect air in the feeding set and the pump dial has not yet been set to **MARCHA (RUN)** after one of the following has occurred:

- The door has opened and closed, or
- The pump dial is turned from **OFF/CARGA (OFF/CHRG)** to any other position.

Otherwise, the pump will begin incremental priming when this button is depressed.

The Options Button is used to change the audio volume of the alarms, as well as the brightness and timing of the LCD display. Press the button until the option you wish to change is displayed, then use the Up and Down Arrow Buttons to choose your preference:

#### **NIVEL SONIDO (AUDIO LEVEL)**

- BAJO (LOW)
- ALTO (HIGH)

#### **BRILLO (BRIGHTNESS)**

- BAJA (LOW)
- MED (MED)
- ALTO (HIGH)

#### **LUZ (LITE- controls when display is on)**

- 10 SEG (10 SEC ON)
- SIEMPRE (ALWAYS ON)

NOTE: "SIEMPRE" ("ALWAYS ON") option is only available when pump is plugged in to live AC/Mains, in order to conserve battery charge.

**COMM** (for "communication", controls which Accessory Cable is enabled for communication)

- NC
- DATA

NOTE: See INSTRUCTIONS FOR USE for complete details

# CLEANING

---

For trouble-free operation, check the pump daily and clean immediately after spills occur. **Before cleaning, be sure that pump is turned OFF and UNPLUGGED.**

**Do not submerge, autoclave, heat, steam, ETO or radiation sterilize the pump.**

- Clean outside surface only (do not spray water or cleaning solution into pump).

## **General Cleaning Instructions:**

- For general cleaning, use soft cloth and warm soapy water. Wipe and dry thoroughly.
- Do not remove the rotor.
  - » After cleaning the rotor, verify that all rollers (on rotor) turn freely.

## **Disinfecting Instructions:**

**If necessary to disinfect pump, please follow the procedure below.**

**Extended exposure to cleaning solutions can cause damage to the pump.**

- Suggested Disinfecting Solutions:
  - » 10% solution of 5.25% sodium hypochlorite (household bleach)
  - » 40% solution of ethyl alcohol
  - » 10% solution of phosphoric acid
  - » For additional disinfecting solutions - contact your local Abbott representative
- Clean pump as instructed in general cleaning instructions
- Apply the disinfectant in the identified formulated mixture
- Let stand for 5 minutes minimum (or longer if specified by the manufacturer of the solution)
- Wipe **thoroughly** with a **damp cloth** and air dry

**NOTE:** These recommendations are not substitutes for official procedures that may differ among institutions.

# PUMP ACCURACY TESTING


There are many variables that can affect flow rate, such as fluid *head height* and fluid *viscosity*. The Abbott FreeGo pump system has been optimized for delivery accuracy when using Abbott Nutrition 1.0 kcal Ready-To-Hang (RTH) feeding formulas. To check flow rate accuracy, use the following procedure:

1. Select a new Abbott FreeGo feeding set for this test. Be sure to use a graduated cylinder for volume measurement—not an enteral container or other uncalibrated vessel.
2. Use an Abbott Nutrition feeding formula in a Ready-To-Hang (RTH) container and set up pump according to INSTRUCTIONS FOR USE.
3. Suspend feed container to side or behind the pump. Recommended head height is 30 cm from the center of the rotor to the top of the fluid in feed container.
4. With the test pump connected to AC/Mains, set the rate to 84 mL/hr.
5. Run fluid into graduated cylinder for 8 hours and compare the accumulated volume in the graduated cylinder to chart below.

<b>FLUJO (SET RATE)</b>	<b>Amount Expected in Graduated Cylinder at 8 hours</b>
84 mL/hr =	638 mL to 706 mL

If the amount in the graduated cylinder is incorrect based on **FLUJO (SET RATE)**, try a new Abbott FreeGo feeding set and repeat the procedure.

The diagram below describes how variation from recommended head height affects flow rate accuracy.

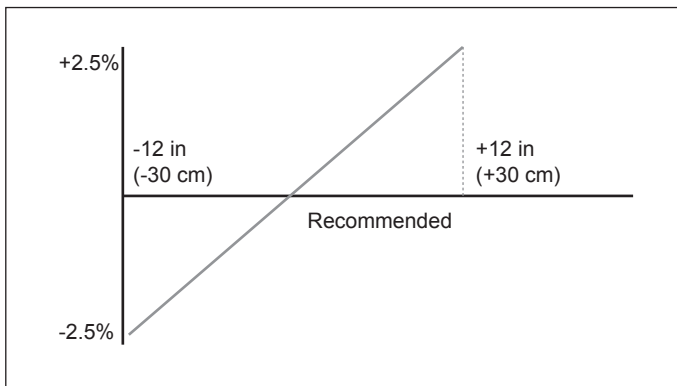## **ACCESSORY INFORMATION**

---

For details of the accessories which are available for the FreeGo enteral feeding pump please contact your local Abbott Nutrition representative.

## **SERVICE INFORMATION**

---

In the event of a pump malfunction or the need for technical assistance or parts, please contact your local Abbott Nutrition representative.

All servicing or adjustments should be performed by Abbott Nutrition-authorized technical personnel.

There are no routine calibrations, adjustments, or services required with the Abbott FreeGo pump.

## **WARRANTY INFORMATION**

---

The Abbott FreeGo pump has a warranty period of 24 months.

## **DISPOSAL INFORMATION**

---

At the end of their service lifetime, the pump and its electrical accessories (AC Adapter Charger, Data Download Accessory Cable, Nurse Call Accessory Cable, and Car Charger) should be disposed of according to local standards and regulations governing the disposal of electronic waste (e-waste). Other accessories can be disposed of or recycled as standard non-hazardous waste.

See the BATTERY section under SPECIFICATIONS for guidance on disposal of the internal pump battery.

# SPECIFICATIONS

---

## **MECHANICAL**

---

Height:	140 mm
Width:	120 mm
Depth:	80 mm
Weight:	490 g

---

## **OPERATIONAL**

---

Flow Rate:	1 mL/hr to 400 mL/hr
Dose:	1 mL to 9999 mL (feed until empty when <b>DOSIS (SET DOSE)</b> at <b>INF DOSIS</b> (infinite dose))
Increments:	1 mL
Priming Rate:	≥ 700 mL/hr
Accuracy:	± 5%, under recommended head height (see PUMP ACCURACY TESTING)
Occlusion Detection Pressure:	Upstream: -34 ± 20 kPa Downstream: 103 ± 20 kPa for 30 seconds 124 ± 20 kPa for 5 seconds
Head height effect:	≤ 2.5% per 30 cm (12 inches) (see PUMP ACCURACY TESTING)
Operating Temperature:	+5° to +40° C
Operating Humidity:	10% to 95% non-condensing
Atmospheric pressure:	65 kPa to 102 kPa
Drop	Dropping from any angle from a height of 1 meter (3 feet) should not damage pump operation.
Programming Modes:	SET RATE, SET DOSE (See INSTRUCTIONS FOR USE)
Operating Mode:	RUN
Alarms:	See ALARM AND WARNING MESSAGES
Infusion sets, models and installation:	See FEEDING SETS

---

## STORAGE

Storage Temperature:	-20° to +65 ° C
Storage Humidity:	10% to 95% non-condensing
Atmospheric pressure:	50 kPa to 106 kPa

## POWER

Voltage:	Charger input 100-240 V; 50-60- Hz; output 5 V DC 2.4A
Power consumption:	320 mW typical (Run Mode, 125 ml/hr)
Type of protection fuse:	Single blow fast acting
Operating voltage fuse:	125 V
Current opening fuse:	2 A
Type of protection against electric shock:	Galvanic isolation by transformer and opto-coupler
Degree of protection against electric shock:	Class II, double insulated
Degree of protection against harmful ingress of water:	IP25
Degree of safety in the presence of a flammable anesthetic mixture:	Possible explosion hazard - Do not use

## BATTERY

Type:	Lithium Ion, 3.7 volt
Charge Life:	≥ 24 hours, at 125 mL/hr
Charge Time:	≤ 6 hours

When the alarm message “**BATT LOW BATTERY**” first appears, there is at least 20 minutes of battery capacity remaining for pump operation.

- Pumps must be charged for a minimum of 6 hours prior to use to ensure optimum performance.
- The FreeGo pump battery may be fully discharged when received and may require approximately 2 minutes of AC power prior to illumination of the AC/Mains LED indicator.

The European Battery Directive requires collection of spent batteries, aiming to facilitate recycling and to protect the environment. This device contains a battery that is not intended to be serviced or removed by the user. The battery in this product should be removed at the end of the life by Abbott Nutrition-authorized technical personnel only, and disposed in accordance with local community regulations for separate collection of spent batteries.

# SYMBOLS

---

IP25

IEC 60529 IP25 degree of protection against fingers (>12 mm) or water entering the enclosure. Water jets from any direction shall have no harmful effect.

---

IPX5  
Type BF

EN 60601-1 Type BF degree of protection against electrical shock. No electrical connection to patient.

---

Not for intravenous (IV) use. Pump is designed for enteral feeding only.

---

The European Battery Directive requires separate collection of spent batteries in accordance with local community regulations.

---

Follow instructions for use.

---

General warning sign.

---

Class II electrical equipment, which is double insulated and does not require a safety connection to earth ground.

---

**Abbott Ireland**  
Ballytivnan  
Sligo  
Ireland

---

Manufactured By:  
**ZEVEX, Inc.**  
4314 Zevex Park Lane  
Salt Lake City, UT 84123 USA

---

# DISTRIBUIDORES - DISTRIBUTORS

---

<b>AU</b> Abbott Australasia PTY. LTD. 299 Lane Cove Road, Macquarie Park, NSW 2113 AUSTRALIA	<b>EL</b> Abbott Laboratories Hellas Vouliagmenis Avenue 512 174 56 Alimos Athens Greece	<b>SA</b> Al-Kamal Import Office Abbott Device Service Center Abraj Atta' awuneya, South Tower, Lower Ground Level King Fahad Road Olaya P.O. Box 20628 Riyadh, 11465 Saudi Arabia
<b>BE</b> Abbott SA/NV Division Nutrition Avenue Einstein 14 1300 Wavre Belgique	<b>ES</b> Abbott Laboratories, S.A. Avenida de Burgos, 91 28050-Madrid	<b>SG</b> Abbott Laboratories (S) Pte Ltd 1 Maritime Square #12-01 Harbourfront Centre Singapore 099253
<b>CA</b> Abbott Laboratories, Limited Laboratoires Abbott, Limitée 8401 TransCanada Highway, Saint-Laurent, Qc, H4S 1Z1 Canada	<b>HR</b> Phoenix Farmacija d.d. Ozaljska 95 10000 Zagreb Croatia	<b>SI</b> Abbott Laboratories d.o.o. Dolenjska c. 242c 1000 Ljubljana, Slovenia
<b>CH</b> Abbott AG Abbott Nutrition International Neuhofstrasse 23 CH-6341 Baar	<b>IE</b> Abbott Laboratories Ireland Ltd. 4051 Kingswood Drive Citywest Business Campus Dublin 24, Ireland.	<b>SK</b> Abbott Laboratories, s.r.o. Karadžičova 10 821 08 Bratislava Slovakia
<b>CL</b> Importado y Distribuido por Abbott Laboratories de Chile Ltda. Av. El Salto #5380 Huechuraba. Santiago	<b>IT</b> Abbott S.r.l. Viale Giorgio Ribotta, 900144 - ROMA ITALIA	<b>TR</b> Abbott Laboratuvarları İth. İhr. ve Tic. Ltd. Şti. Ekinçiler cad. Hedef Plaza No:3 Kavacık-Beykoz 34810 İstanbul - Türkiye
<b>CO</b> ABBOTT LABORATORIES DE COLOMBIA S.A. Calle 100 No. 9A-45. Bogotá D.C., Colombia.	<b>NL</b> Abbott BV Abbott Nutrition Wegalaan 9 2132 JD Hoofddorp Nederland	<b>UK</b> Abbott Laboratories U.K., Ltd. Queenborough, Kent ME11 5EL
<b>CZ</b> Hadovka Office Park Evropská 2590/33d 160 00 Prague 6 Czech Republic	<b>NZ</b> Abbott Laboratories NZ Ltd. Building D, 4 Pacific Rise Mt. Wellington AUCKLAND NEW ZEALAND	
<b>DE</b> Abbott Nutrition International Abbott GmbH & Co.KG Max-Planck-Ring 2 65205 Wiesbaden	<b>PE</b> Abbott Laboratorios S.A. Av. Brasil 2730, Lima 21 - Perú. R.U.C. No. 20100096936	

# REFERENCIAS - STANDARDS

- Directive 93/42/EEC of 14 June 1993 concerning medical devices, as amended.
- IPX1: Protected against dripping water (Accessory cables).
- IP25: IEC 60529 IP25 degree of protection against fingers (>12 mm) or water entering the enclosure. Water jets from any direction shall have no harmful effect.
- EN 60601-1 (Ed. 3) Medical electrical equipment - Part 1: General requirements for basic safety and essential performance.
- EN 60601-1-2 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral standard: Electromagnetic compatibility - Requirements and tests.
- EN 60601-1-6 Medical electrical equipment - Part 1-6: General requirements for basic safety and essential performance - Collateral standard: Usability.
- EN 60601-1-8 Medical electrical equipment - Part 1-8: General requirements for basic safety and essential performance - Collateral Standard: General requirements, tests and guidance for alarm systems in medical electrical equipment and medical electrical systems.
- EN 60601-1-11 Medical electrical equipment – Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment.
- EN 61000-3-2 Electromagnetic compatibility (EMC) - Part 3-2: Limits - Limits for harmonic current emissions (equipment input current  $\leq 16$  A per phase).
- EN 61000-3-3 Electromagnetic compatibility (EMC) - Part 3-3: Limits - Limitation of voltage changes, voltage fluctuations and flicker in public low-voltage supply systems, for equipment with rated current  $\leq 16$  A per phase and not subject to conditional connection.
- EN 62366: Medical devices -- Application of usability engineering to medical devices.

Guidance and manufacturer's declaration – electromagnetic emissions		
The Abbott FreeGo Pump is intended for use in the electromagnetic environment specified below. The customer or the user of the Abbott FreeGo Pump should assure that it is used in such an environment		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Abbott FreeGo Pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	The Abbott FreeGo Pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

## Guidance and manufacturer's declaration – electromagnetic immunity

The Abbott FreeGo pump is intended for use in the electromagnetic environment specified below. The customer or the user of the Abbott FreeGo pump should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact  ± 8 kV air	± 6 kV contact  ± 8 kV air	For best protection from ESD, avoid using the Abbott FreeGo pump in environments that promote high levels of static discharge (e.g. synthetic floor materials in low humidity environments).
Electrostatic fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines  ± 1 kV for input/output lines	± 2 kV for power supply lines  ± 1 kV for input/output lines	Mains power quality should be that of a typical hospital, acute care, long term care, or residential environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s)	± 1 kV differential mode	Mains power quality should be that of a typical hospital, acute care, long term care, or residential environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % $U_T$ (>95 % dip in $U_T$ ) for 0.5 cycle  40 % $U_T$ (60 % dip in $U_T$ ) for 5 cycles  70 % $U_T$ (30 % dip in $U_T$ ) for 25 cycles  <5 % $U_T$ (>95 % dip in $U_T$ ) for 5 s	<5 % $U_T$ (>95 % dip in $U_T$ ) for 0.5 cycle  40 % $U_T$ (60 % dip in $U_T$ ) for 5 cycles  70 % $U_T$ (30 % dip in $U_T$ ) for 25 cycles  <5 % $U_T$ (>95 % dip in $U_T$ ) for 5 s	Mains power quality should be that of a typical hospital, acute care, long term care, or residential environment. The Abbott FreeGo pump allows continued operation during power mains interruptions via the internal battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	Class 2 (3 A/m)	Power frequency magnetic fields should be at levels characteristic of a typical location in the typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

## Guidance and manufacturer's declaration – electromagnetic immunity

The Abbott FreeGo pump is intended for use in the electromagnetic environment specified below. The customer or the user of the Abbott FreeGo pump should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Conducted RF	3 Vrms	3 Vrms	<p>Avoid using the Abbott FreeGo pump in close proximity to electronic devices or equipment that emit radio-frequency (RF) energy, such as X-ray and magnetic resonance imaging equipment. Actively transmitting cell phones and similar devices should be kept two or more meters away from the pump.</p> <p><b>Technical Guidance</b> Portable and mobile RF communications equipment should be used no closer to any part of the Abbott FreeGo pump, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance</b></p> $d = 1.2\sqrt{P}$
IEC 61000-4-6	150 kHz to 80 MHz	3 V/m	$d = 1.2\sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2.3\sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$
Radiated RF	3 V/m		<p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> <div style="text-align: center;">  </div>
IEC 61000-4-3	80 MHz to 2.5 GHz		

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Abbott FreeGo pump is used exceeds the applicable RF compliance level above, the Abbott FreeGo pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Abbott FreeGo pump.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile

RF communications equipment and the Abbott FreeGo pump

The Abbott FreeGo pump is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Abbott FreeGo pump can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Abbott FreeGo pump as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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## **UK**

Abbott Nutrition, Abbott House  
Vanwall Business Park  
Vanwall Road, Maidenhead  
Berkshire SL6 4XE, UK  
www.abbottnutrition.co.uk.

**Abbott**

A Promise for Life

44838 Rev. J

Rev. July/15

CE  
0123

 **Abbott**  
A Promise for Life