GoodKnight® 425

GoodKnight® 425 ST

**Practitioner and
Service Provider manual**
**Manual de uso del proveedor de
servicios domiciliarios**
**Manual do Médico
e do Prestador de Serviço**

Manufacturer
Fabricante, Fabricante

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United Kingdom

Reino-Unido

Tyco Healthcare
Technical Department
Unit 2
Talisman Business Centre
London Road
Bicester
Oxfordshire OX26 6HR
Tel. +44 01869 328092

GoodKnight 425 / GoodKnight 425 ST

Practitioner and Service Provider manual	1
Manual de uso del proveedor de servicios domiciliarios	15
Manual do Médico e do Prestador de Serviço	29

GoodKnight® 425
GoodKnight® 425 ST

**Practitioner and
Service Provider manual**

List of revisions for the Practitioner and Service Provider manual

GoodKnight® 425 / GoodKnight® 425 ST

The pages below are included in the Practitioner and Service Provider manual
Ref. M-145DFU00-30 revision A.

Current pages	Revision
Cover	A
List of revisions	A
Table of contents	A
Introduction	A
1 – 13	A

Reference or revision	Description	Date
M-145DFU00-30 Revision A.	Creation of the manual	July 2004

© 2004 Puritan Bennett
All rights reserved.

The information contained in this manual is a supplement to the **GoodKnight 425 / GoodKnight 425 ST** patient manual (GK425/GK425 ST).

This manual is intended for the use of the Practitioner and the Service Provider. Under no circumstances should it be supplied to the patient.

For all information related to the installation of the device and its accessories, please refer to the patient manual.

The service provider may enter notes in the space provided in the back of the patient manual.

All the information contained in this manual is valid for devices *GoodKnight 425* and *GoodKnight 425 ST*, unless stated otherwise.

The *GoodKnight 425* device may be connected to a computer to adjust its settings and recover data stored in its memory. The **Silverlining**[®] software is practical and user-friendly. It easily displays, analyses and prints the collected data. For more information about the software use, please refer to its separate user manual.

GoodKnight[®] is a registered trademark of Tyco Healthcare.

This page is left blank intentionally

TABLE OF CONTENTS

BEFORE STARTING	3
SAFETY INSTRUCTIONS	3
INDICATIONS FOR USE	3
PRECAUTIONS OF USE	3
ACCESSING THE SETTINGS OF THE DEVICE	4
DEFINITIONS	4
SETTING RANGES	5
SETTINGS	6
LIST OF AUTHORISED ACCESSORIES	11
MASKS	11
HOSES	11
OTHER ACCESSORIES	11
TECHNICAL CHARACTERISTICS	13
DISPOSAL OF THE DEVICE AT THE END OF ITS LIFE	13

This page is left blank intentionally

BEFORE STARTING

Safety instructions

In order to ensure safe and efficient device usage, read this operating manual as well as page 1 in the patient manual. Please pay special attention to the passages preceded by the following terms:

WARNING

There may be a danger of death or serious injury to the patient.

CAUTION

There is a possibility of material damage to this or any other device.

NOTE

Indicates special points aimed at more efficient and practical operation.

WARNINGS

1. Read the general warnings given in the patient manual.
2. Only use accessories authorised by the manufacturer, as listed in this manual.
3. The patient **should** receive the patient manual, **as well as** the instructions for using the accessories.
4. Configure the *GoodKnight 425* device according to the instructions given in this manual in order to be able to use it under optimum conditions of safety and operation.

Indications for use

The Puritan Bennett *GoodKnight 425* is indicated for use in treating Obstructive Sleep Apnea (OSA) in spontaneous breathing patients weighing over 30 kg within a homecare and hospital environment.

Precautions of use

The *GoodKnight 425* device must be used with caution in patients presenting with the following conditions:

- Pneumothorax or pneumomediastinum
- Severe bullous emphysema or previously complicated by pneumothorax
- Massive epistaxis or previous massive epistaxis
- Pneumocephalus, trauma or recent surgery with the after-effect of a cranio-nasopharyngeal fistula
- Acute sinusitis, otitis media or perforation of the tympanic membrane.

ACCESSING THE SETTINGS OF THE DEVICE

The *GoodKnight 425* device detects the respiratory cycles of the patient and, in response to its inspiratory and expiratory efforts, delivers two positive pressure levels, a high level of inspiratory pressure IPAP and a low level of expiratory pressure EPAP.

If the inspiratory pressure level IPAP is equal to the expiratory pressure level EPAP, then the device generates a continuous positive pressure during all the respiratory cycle.

The *GoodKnight 425* device operates without Back-Up rate, whereas the *GoodKnight 425 ST* device can operate with or without Back-Up rate.

The settings parameters are accessible, either directly on the device, or via the computer by using the SilverLining software.

WARNING

Before setting the device, check that it delivers an airflow when it is in operation mode (see chapter titled “Use” in the patient manual). In the opposite, stop it immediately and contact the relevant technical department, listed at the beginning of the manual.

Definitions

Positive Inspiratory Pressure level (IPAP=Inspiratory Positive Airway Pressure) :

High pressure level for the inspiration.

Positive Expiratory Pressure level (EPAP=Expiratory Positive Airway Pressure) :

Low pressure level for the expiration.

Ramp time:

Period of time over which the pressure gradually increases from the starting ramp pressure to the reference pressure levels with ramp activation.

Starting ramp pressure:

Pressure level at the start of ramp.

Session:

Period during which the data are recorded in the device memory between start-up and standby. During a session, all the data are collected and displayed in a storage block (the data cannot be displayed for a session part). A session is recorded if the device is on for 15 minutes without breaking off or after ramp time.

I/E ratio:

Ratio between the inspiratory time and the expiratory time.

Back-Up rate (on the *GoodKnight 425 ST* device only):

The Back-Up rate corresponds to the respiratory cycle which will be provided when the device does not detect the patient respiratory cycle or when the patient spontaneous breathing is lower than the prescribed Back-Up rate.

Setting ranges

For each parameter, the following table shows the minimum, maximum and default values which can be set:

Parameters	Minimum value	Maximum value	Default value	Step
GoodKnight 425				
• IPAP level	3 cmH ₂ O	25 cmH ₂ O	15 cmH ₂ O	0,5 cmH ₂ O
• EPAP level	3 cmH ₂ O	IPAP or 20 cmH ₂ O	5 cmH ₂ O	0,5 cmH ₂ O
• Ramp time	0 mn	30 mn	15 mn	5 mn
• Starting ramp pressure	3 cmH ₂ O	EPAP	3 cmH ₂ O	0,5 cmH ₂ O
• Rise time	1 (about 300 ms)	5 (about 600 ms)	3 (about 450 ms)	1
• Inspiratory sensitivity	1	10	5	1
• Expiratory sensitivity	1	10	5	1
GoodKnight 425 ST only				
• Back-Up rate	0 or 4 bpm	25 bpm	10 bpm	1 bpm
• I/E ratio	1/4	1/0,5	1/2	1/0,5


NOTE

The compensation in altitude (compared to the pressure) is carried out automatically by a pressure sensor.

Settings

In order to restrict access of certain settings to the medical team, a hidden key has been provided on the device. It is located under the letters “Go” in the “*GoodKnight*” name. This is detectable to the touch as a hollow.

The parameter values may be changed by using the on/off and ramp keys as shown in the figure “View of the control buttons” below:

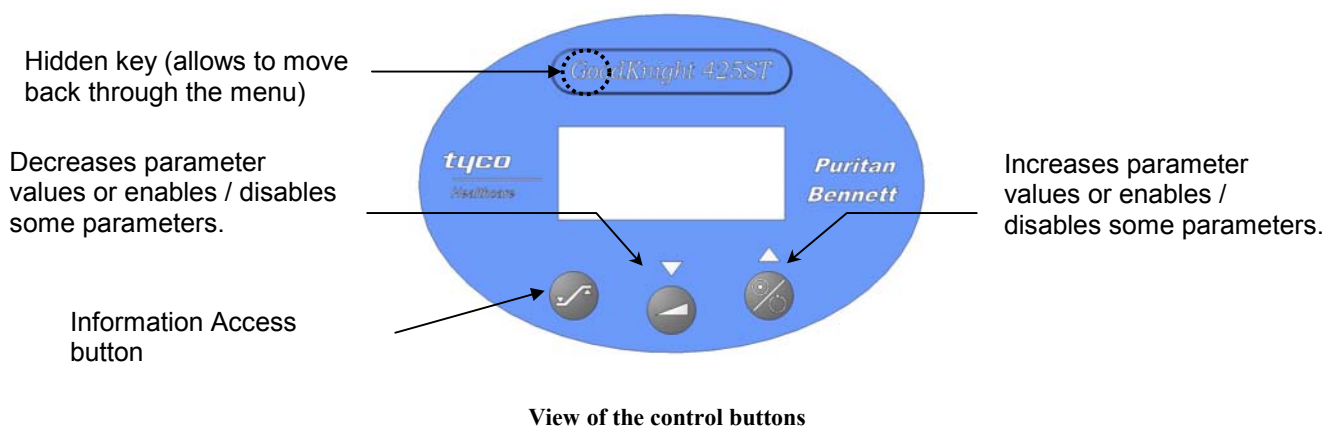NOTE


The layout of the buttons is the same on the *GoodKnight 425 ST* as on the *GoodKnight 425*, only the device's name is different.

To access all the settings, press the hidden key **for one second** and then **simultaneously** the information access key .

NOTES

- The symbol  reminds you that the parameter value is adjustable.
- All the values displayed in the following screens are given as a guide only.

Press the hidden key for one second and  key .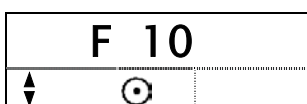

Back-up rate of the device.

NOTE


This function is accessible only on the *GoodKnight 425 ST* device.

Use the on/off or ramp key to increase or decrease the Back-up rate value. This value is adjustable between 4 and 25 bpm.

If 0 is displayed, it means that there is no Back-up rate.

With each additional press of the information access key, the following settings will be available.


Press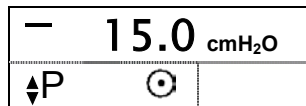

Positive Inspiratory Pressure IPAP level set of your device.

You can increase or decrease the positive inspiratory pressure level of the device by using the on/off or ramp key. This value is adjustable between 3 cmH₂O and 25 cmH₂O.


Press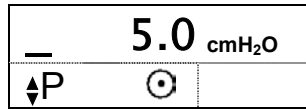

Positive Expiratory Pressure EPAP level set of your device.

You can increase or decrease the positive expiratory pressure level of the device by using the on/off or ramp key. This value is adjustable between 3 cmH₂O and the IPAP value or 20 cmH₂O at the maximum.


Press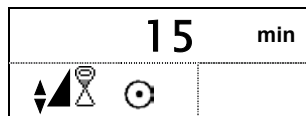

Ramp time.

You can decrease or increase the value displayed up to 30 minutes by using the ramp (down) or on/off (up) key.

Press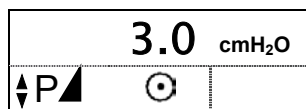


Starting ramp pressure (pressure level at the start of ramp); this parameter is adjustable and visible only if the ramp time value is greater than 0.

The value displayed can be decreased or increased, between 3 cmH₂O and the EPAP value, by using the ramp (down) or on/off (up) key.

For further details, refer to the chapter titled “Using the ramp” in the patient manual.

Press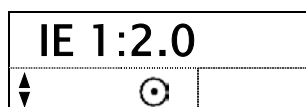

I/E ratio (ratio between Inspiratory time / Expiratory time); this parameter can be displayed only if the Back-Up rate is greater than 0.


NOTE

This function is accessible only on the *GoodKnight 425 ST* device.

The value displayed can be increased or decreased by using the on/off (up) or ramp (down) key.

However, certain conditions of Back-Up rate and pressures can limit the possible extent of this parameter adjustment.


Press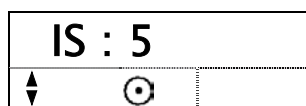

Inspiratory sensitivity.

The value displayed, adjustable between 1 and 10 (1 is the more sensitive setting), can be increased or decreased by using the on/off (up) or ramp (down) key.


Press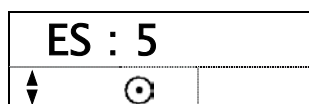

Expiratory sensitivity.

The value displayed, adjustable between 1 and 10 (1 is the more sensitive setting), can be increased or decreased by using the on/off (up) or ramp (down) key.

Press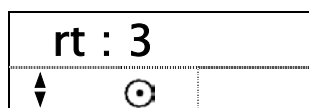

Pressure rise time (period of time over which the pressure increases from the EPAP level to the IPAP level). It can be decreased or increased, between 1 (about 300 ms) and 5 (about 600 ms), by using the ramp (down) or on/off (up) key.

However, certain conditions of Back-Up rate, pressures and I/E ratio can limit the possible extent of this parameter adjustment.

Press


Hour meter value (device operating time). The value displayed on the right below represents the minutes.

NOTE

The hour meter value cannot be changed.

Press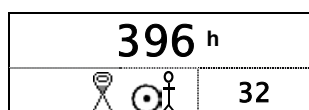


Use meter value (time after the ramp time or 15 minutes during which the patient has breathed with the mask). The value displayed on the right below represents the minutes.

NOTE


The use meter value cannot be changed.

Press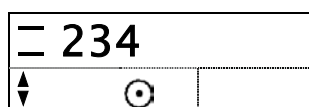

Offset of the pressure sensor integrated into the *GoodKnight 425* device. You can increase or decrease the value displayed by using the on/off (up) or ramp (down) key. This value can be adjustable between 0 and 255. To set it, refer to the section “Adjustment of the pressure at the device outlet” on page 9.

Press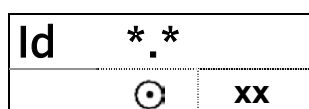

Software version.


The version number is shown in two digits (*), for example the screen may display : Id 1.0 and the value displayed on the right below represents the motor driver version.

CAUTION

If, when in use, the device stops and displays the message **Er 1**, this means a problem was detected in the settings stored. In this case, press the information access button, the device displays the « dEF » blinking message, then press the on/off button. The machine is then automatically reset to the default values and you must set the desired parameters again.

To exit the parameter setting menu, press again the information access button.

In this case, one of the following two screens indicates the status of your equipment (the values displayed are given as a guide only).

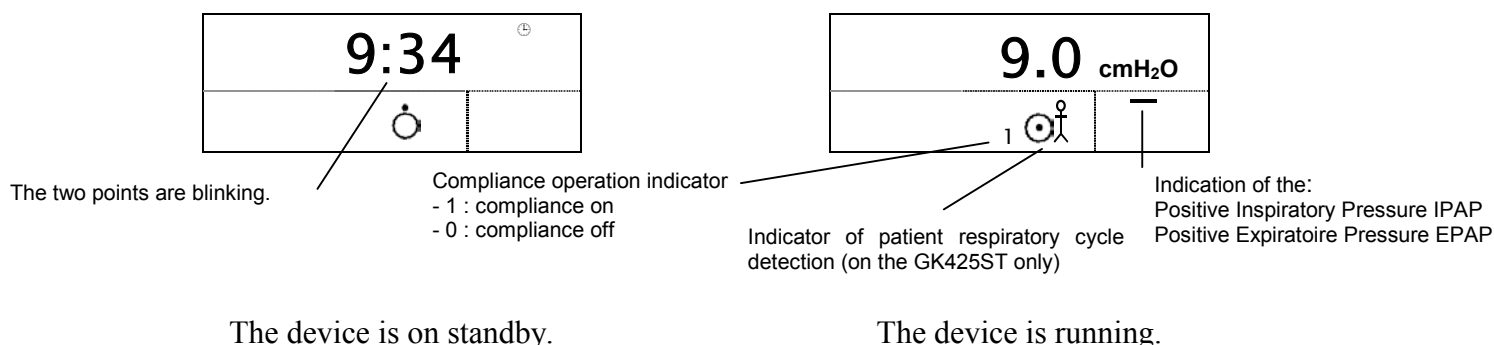NOTE

Exit from the parameter setting menu takes place automatically when you do not press any key for 30 seconds.

Adjustment of the pressure at the device outlet


Connect the spiral hose to the device and the small hose that emerges through the spiral hose to the pressure point. Connect the hose's free nozzle to a plug equipped with a 4 mm leak and a pressure point. Turn the device blower on, then connect the pressure point to a manometer and adjust the pressure order to 14 cmH₂O. Access to the pressure sensor offset setting as described in the section titled « Settings » of this manual. Adjust the value by using the on/off (up) or ramp (down) key in order to read a pressure equal to 14,0 cmH₂O on the manometer. Then, while maintaining the pressure on the hidden key, press the information access key. The pressure sensor offset setting is automatically memorized.

Change of unit of pressures displayed

The unit of the pressures displayed may be changed between cmH₂O and hPa, and conversely. To change it, press and hold the hidden key while you connect the device to the electrical outlet (by the power supply module). The machine displays the new unit selected until you release the pressure on the hidden key. To change again the unit displayed, repeat the process.

Setting the time

You can set the time only when the device is on standby. To do so, press the ramp button  for one second, and then the on/off button  at the same time. If you release the pressure on the buttons, the symbol of adjustment  appears under the hour, as shown on the screen hereafter:

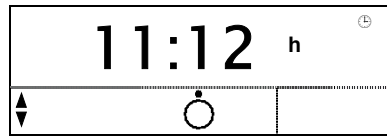The letter h indicating the hour blinks, you can then increase and decrease it using the ramp and on/off keys.

To set the minutes, press the information access button, the minutes symbol blinks and you can set them by following the same procedure as for the hour.

Press the information access button again to exit the menu.

NOTE

Exit from the parameter setting menu takes place automatically when you do not press any key for 30 seconds.

To set the date, please refer to the SilverLining with *GoodKnight 425* user manual.

LIST OF AUTHORISED ACCESSORIES

You will find below the accessories which can be used with the *GoodKnight 425* device.

Masks

Nasal+confort mask with 4 mm calibrated vent (6 sizes):

Ref. M-2680x0-05 where x varies from 2 to 7.

Nasal+confort mask without 4 mm calibrated vent (6 sizes):

Ref. M-2680x0-06 where x varies from 2 to 7.

Nasal+ mask with 4 mm calibrated vent (6 sizes):

Ref. M-2680x0-02 where x varies from 2 to 7.

Nasal+ mask without 4 mm calibrated vent (6 sizes):

Ref. M-2680x0-04 where x varies from 2 to 7.

WARNING

The Nasal+ and Nasal+confort masks without vent must be used with the Pirouette Port.

Nasal interface Breeze:

Ref. Y-101400-00.

DreamSeal™ mask:

Ref. Y-103074-00A ; Y-102616-00 ; Y-103059-00A.

Hoses

Adam hose:

Ref. S-133293-00 ; S-133294-00.

22mm Ø hose:

Ref. M-261000-27 (1,80 m); M-261000-30 (2,70 m).

Other accessories

Oxygen adaptor:

Ref. Y-616155-03.

Pirouette port:

Ref. M-368010-00 (x1), M-368011-00 (x5).

Heated humidifier:

Fisher & Paykel HC100 and HC150.

Heated humidifier GoodKnight H₂O:

Ref. M-114600-UK.

Adam Pillows (7 sizes):

Ref. S-616323-00A, S-616324-00C, S-616325-00A, S-616355-00C, S-616356-00C, S-616357-00C, Y-101969-00.

Air inlet filter:

Ref. M-413950-02.

Thin filter:

Ref. M-413950-04.

Serial connection cable:

Ref. M-213930-04.

Battery cable:

Ref. M-214530-00

Battery cable/cigarette lighter plug:

Ref. M-214530-01

Carrying case:

Ref. M-213930-01

Kit modem Ref. M-213930-12 including:

GK418 modem cable Ref. M-213530-11

GK420 modem adapter Ref. M-213930-05

TECHNICAL CHARACTERISTICS

Please refer to the *GoodKnight 425* patient manual to know the device characteristics.

Disposal of the device at the end of its life

In order to protect the environment, the different parts of the equipment and its accessories should be disposed of in approved dumps in conformity with the regulations in force.

Device complies with the requirements of the directive 93/42/CEE related to medical appliances.

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